

DISINFECTANTS

AMITY PAA 5 SYSTEM

PERACETIC ACID BASED HIGH LEVEL DISINFECTANT

High-level disinfectant for semi-critical thermolabile medical devices such as endoscopes and their accessories.

Amity PAA 5 is a powerful peracetic acid based high level disinfectant concentrate for use in Automated Endoscope Reprocessors (AERs).

It is an acidic disinfection concentrate for medical devices.

Amity PAA 5 System Part A and B are combined to produce a single use solution for HLD (High Level Disinfection).

Amity PAA 5 System is registered as Class IIB Medical Device



Primary features attributable to Amity PAA 5 System are:

- Active ingredient: Peracetic Acid (PAA).
- Dilution Required: 20ml/L of parts A&B.
- Contains Corrosion Inhibitors.
- High Level Disinfectant: 10 minutes at 35°C, clean conditions.
- Tested to EN 14885:2022 for disinfectant properties (bactericidal, fungicidal, mycobactericidal, virucidal and sporicidal against C. diff.).
- 2-part product, no activation required.
- Contains no aldehydes.

Packaging

- 5 litre containers
- 10 litre containers



Amity PAA 5 System - A Peracetic Acid based High Level Disinfectant for Medical Devices

DISINFECTANTS

Application

For disinfection:

Ensure instructions for use (IFU) provided by the manufacturer of both the medical device and AER manufacturer (as required by EN ISO 17664) are followed.

Ensure pre-cleaning of medical devices prior to disinfection.

Amity PAA 5 System Parts A and B dosed together at 20ml/L of each at 35°C for a contact time of 10 minutes.

AMITY PAA 5 SYSTEM PERACETIC ACID BASED HIGH LEVEL DISINFECTANT

Amity PAA 5 System Properties

Use Recommendation:

Conditions:	Clean
Dilution:	20ml/L Part A and B
Temperature:	35°C
Contact time:	10 minutes

Activity	Standards	Organism
Bactericidal	EN 13727 EN 14561	<i>Pseudomonas aeruginosa</i> <i>Staphylococcus aureus</i> <i>Enterococcus hirae</i>
Fungicidal	EN 13624 EN 14562	<i>Candida albicans</i> <i>Aspergillus brasiliensis</i>
Virucidal	EN 17111	Human adenovirus Murine norovirus
	EN 14476	Human adenovirus Murine norovirus Polio Virus
Mycobactericidal	EN 14348 EN 14563	<i>Mycobacterium avium</i> <i>Mycobacterium terrae</i>
Sporicidal against <i>C. diff</i>	EN 17126	<i>Clostridium difficile</i>

Active ingredients: Peracetic Acid (PAA)



Amity Limited, Libra House, West Street,
Worsbrough Dale, Barnsley, South Yorkshire,
S70 5PG, UK

T: +44 (0) 1226 770787

E: sales@amityinternational.com

WWW.AIHCO.CO.UK

© Amity Limited all rights reserved 2023

DISTRIBUTED BY:

Version: 251029