

## DISINFECTANTS

# AMITY PAA 15

## SINGLE PROCESS PERACETIC ACID, NO ACTIVATOR REQUIRED

Amity PAA 15 is a powerful peracetic acid based high level disinfectant / sterilant concentrate for use in Automated Endoscope Reprocessors (AERs). It is an acidic disinfection concentrate for flexible, rigid endoscopes and medical devices.

Amity PAA 15 is registered as Class IIB Medical Device

Amity PAA 15 is a HLD fluid effective against, spores, mycobacteria, viruses, bacteria and fungi when used at a concentration of 1% at 20 °C within a 10 minute exposure cycle.

Devices and endoscope parts should be cleaned prior to disinfection with Amity PAA 15 using suitable cleaners such as Viruzyme family or Virudet N.



### Primary features attributable to Amity PAA 15 are:

- Powerful peracetic acid based high level disinfectant / sterilant (sporicidal, mycobactericidal, virucidal, bactericidal and fungicidal).
- Recommended dilution 1% in water.
- Single component & does not require activator compounds.
- Contains anti-corrosion additives.
- Rapid & wide ranging activity including spores.
- Effective at temperatures down to 25°C within a 10 minute exposure.
- Tested to EN 14885 for disinfectant properties.
- Tested to EN 15883-4.
- Contains no aldehydes and formaldehyde.

### Packaging

- 5 litre containers



**Amity PAA 15 - A Peracetic Acid based High Level Disinfectant (HLD) / Sterilant Fluid for Medical Devices & Instruments**

## DISINFECTANTS

### Application

For disinfection of Medical Instruments and Endoscopic Equipment

#### Stage 1:

Wash medical device carefully in an enzymatic cleaner (Viruzyme Family or Virudet N) and rinse.

#### Stage 2:

Amity PAA 15 should be dosed at 10ml/L into AER at  $\geq 20^{\circ}\text{C}$ , contact time – 10 minutes.

or

Amity PAA 15 should be dosed at 10ml/L into AER at  $\geq 35^{\circ}\text{C}$ , contact time – 5 minutes

## AMITY PAA15

SINGLE PROCESS PERACETIC ACID, NO ACTIVATOR REQUIRED

### Amity PAA 15 Properties

Active Against	Standards	Contact Time
Spores	EN13704 EN14347	10 Minutes 10 Minutes
Mycobacteria	EN14348 Mycobacterium terrae Mycobacterium avium EN14563	10 Minutes 10 Minutes
	Mycobacterium terrae Mycobacterium avium	10 Minutes 10 Minutes
Bacteria	EN13727, EN14561	5 Minutes 5 Minutes
Viruses	EN 14476 Polio & Adeno HIV, Hep B & C	20 Minutes 5 Minutes
Fungi / Yeasts	EN13624 EN14562	5 Minutes 5 Minutes



Amity Limited, Libra House, West Street,  
Worsbrough Dale, Barnsley, South Yorkshire,  
S70 5PG, UK

T: +44 (0) 1226 747660

F: +44 (0) 1226 291936

E: sales@amityinternational.com

[WWW.AIHC.CO.UK](http://WWW.AIHC.CO.UK)

© Amity Limited all rights reserved 2018

DISTRIBUTED BY: