

## DISINFECTANTS


# AMITY PAA 5 SYSTEM

## PERACETIC ACID BASED HIGH LEVEL DISINFECTANT / STERILANT FOR FLEXIBLE ENDOSCOPES & MEDICAL DEVICES

Amity PAA 5 is a powerful peracetic acid based high level disinfectant / sterilant concentrate for use in Automated Endoscope Reprocessors (AERs) and other automated decontamination systems. It is an acidic disinfection concentrate for flexible endoscopes and medical devices.

Amity PAA 5 Solutions A and B are combined to produce a single use solution for HLD (High Level Disinfection) / Sterilization of flexible endoscopes with Peracetic Acid.

Amity PAA 5 System are registered as Class IIB Medical Device

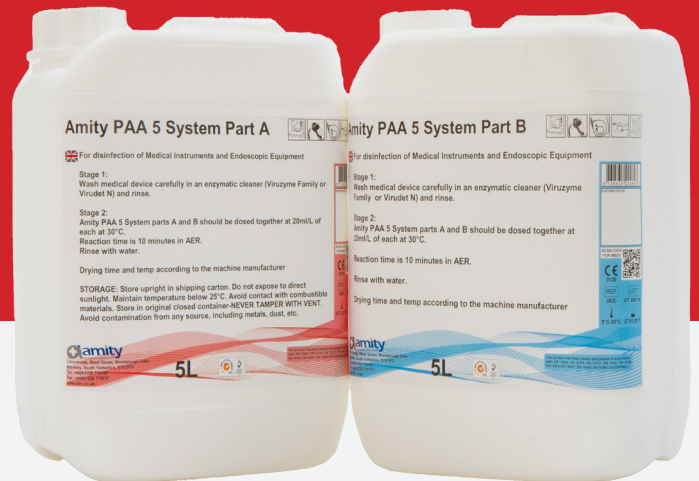

### Primary features attributable to Amity PAA 5 System are.

- Active ingredient: Peracetic Acid (PAA).
- HLD claim: 5 minutes at 30°C.
- Dilution Required: 1:1 to 48 water.
- Contains Corrosion Inhibitors
- Added Water Softeners.
- Tested to EN 14885 for disinfectant properties.
- Tested to EN 15883-4.

### Packaging

- 5 litre containers



**Amity PAA 5 System - A Peracetic Acid  
based High Level Disinfectant for Flexible  
Endoscopes & Medical Devices**

## DISINFECTANTS

### Application

For disinfection of Medical Instruments and Endoscopic Equipment

#### Stage 1:

Wash medical device carefully in an enzymatic cleaner (Viruzyme Family or Virudet N) and rinse.

#### Stage 2:

Amity PAA 5 System parts A and B should be dosed together at 20ml/L of each at 30°C.

Reaction time is 5 minutes in AER.

Rinse with water.

## AMITY PAA 5 SYSTEM

PERACETIC ACID BASED HIGH LEVEL DISINFECTANT / STERILANT  
FOR FLEXIBLE ENDOSCOPES & MEDICAL DEVICES

### Amity PAA 5 System Properties

Active Against	Standards	Contact Time
Spores	EN13704 EN14347	5 Minutes 5 Minutes
Mycobacteria	EN14348 Mycobacterium terrae Mycobacterium avium EN14563	5 Minutes 5 Minutes
	Mycobacterium terrae Mycobacterium avium	5 Minutes 5 Minutes
Bacteria	EN13727, EN14561	5 Minutes 5 Minutes
Viruses	EN 14476 Polio & Adeno HIV, Hep B & C	20 Minutes 5 Minutes
Fungi / Yeasts	EN13624 EN14562	5 Minutes 5 Minutes



Amity Limited, Libra House, West Street,  
Worsbrough Dale, Barnsley, South Yorkshire,  
S70 5PG, UK

T: +44 (0) 1226 747660

F: +44 (0) 1226 291936

E: sales@amityinternational.com

[WWW.AIHC.CO.UK](http://WWW.AIHC.CO.UK)

© Amity Limited all rights reserved 2018

DISTRIBUTED BY: