

DISINFECTANTS

SANITAB

EFFERVESCENT CHLORINE TABLETS WITH OR WITHOUT DETERGENT

Since many authorities have banned the use of bleach, NaDCC powders and tablets predominate the favoured method of chlorine disinfection. One of the greatest offenders involved in accidents is liquid bleach, or sodium hypochlorite (NaOCl). Sanitab prevents any accidents happening. Its safe screw cap tub houses the tablets for up to five years. These small, white tablets are based on a dry chlorine donor, troclosene sodium (NaDCC), which is blended with effervescent components before being formed into tablets.

The result is a fast dissolving, highly convenient, safer and more accurate alternative to liquid bleach. NaDCC is well established in the cleaning industry for both hospital and janitorial applications.



Packaging

Sanitab

- 100 Tub (2.5g tablet)
- 100 Tub (4.72g tablet)
-

Sanitab Plus

- -200 Tub (3.25g tablet)



Guaranteed protection and performance, in a simple, convenient, easy to use tablet form

DISINFECTANTS

Application

Sanitab (4.72g tablet)

Disinfection Directions (dissolve tablets in clean water)

Body fluid spills 10,000ppm	Dissolve 7 tablets in 1 litre. Minimum 2 minutes contact time.
Laboratory discard jars 2,500ppm	Dissolve 7 tablets in 5 litres. Soak overnight.
General disinfection 1,000ppm	Dissolve 2 tablet in 3 litres. 15 minutes contact time.
Stainless steel instruments 600ppm	Dissolve 2 tablets per 5 litres. Immerse for 1 hour.
WC's, drains, sinks 400ppm	1 tablet per 3.5 litres. Pour in solution at quiet periods.
Food preparation surfaces, floors, tiles 200ppm	1 tablet in 7.5 litres. Allow to soak for 3 minutes and air dry or use disposable paper towels. Neutral (non cationic) detergent can be added to solution to aid cleaning.
Dishcloths, mops, etc. 60ppm	1 tablet in 25 litres. Soak to bleach clean and deodorise. Do not soak overnight.

Sanitab (2.5g tablet)

Disinfection Directions (dissolve tablets in clean water)

Body fluid spills 10,000ppm	Dissolve 10 tablets in 1 litre. Minimum 2 minutes contact time.
Laboratory discard jars 2,500ppm	Dissolve 5 tablets in 2 litres. Soak overnight.
General disinfection 1,000ppm	Dissolve 1 tablet in 1 litre. 15 minutes contact time.
Stainless steel instruments 600ppm	Dissolve 3 tablets per 5 litres. Immerse for 1 hour.
WC's, drains, sinks 400ppm	1 tablet per 2.5 litres. Pour in solution at quiet periods.
Food preparation surfaces, floors, tiles 200ppm	1 tablet in 5 litres. Allow to soak for 3 minutes and air dry or use disposable paper towels. Neutral (non cationic) detergent can be added to solution to aid cleaning.
Dishcloths, mops, etc. 60ppm	1 tablet in 16 litres. Soak to bleach clean and deodorise. Do not soak overnight.

Sanitab Plus (3.25g tablet)

Detergent sanitiser tablets provide a powerful disinfectant with a foaming anionic surfactant that will clean / disinfect surfaces at the same time.

General Cleaning / Disinfecting

*Note: Heavily soiled surfaces should be cleaned first using a neutral detergent.

Add 1 tablet per 5 litres of water to make a solution of 200ppm. Warm water and stirring will speed up the dissolution time. If a more concentrated solution is required, more tablets can be added each one providing 200ppm.

Mop Head Cleaning / Disinfecting

Rinse mop in clean water before chemical disinfection by adding 1 tablet per 5 litres of water. Soak the mop in the solution for 30 minutes. Rinse and store dry.

*Note: Mops used in high bacteria risk areas, add 1 tablet per 1 litre of water to make a solution of 1000ppm.

Environmental Disinfection and Terminal Cleans

Dissolve 1 tablet in 1 litre of water to make a solution of 1000ppm.

SANITAB
EFFERVESCENT CHLORINE TABLETS
WITH OR WITHOUT DETERGENT

Sanitab Properties

Feature		Feature	
Bactericidal Gram +ve	●●●	Bactericidal Gram -ve	●●●
Virucidal	●●●	Sporicidal	●●
Mycobactericidal	●●●	Fungicidal	●●
Resistance to inactivation by organic load	●●	Compact storage, cost of distribution	●●●
Stability in working solution	●●	Metallic corrosion resistance	●●
Ease of preparation	●●●		

●●● Excellent ●● Good ● Fair



Amity Limited, Libra House, West Street,
Worsbrough Dale, Barnsley, South Yorkshire,
S70 5PG, UK

T: +44 (0) 1226 747660

F: +44 (0) 1226 291936

E: sales@amityinternational.com

WWW.AIHC.CO.UK

© Amity Limited all rights reserved 2018

DISTRIBUTED BY: