


DISINFECTANTS

VIRUSOLVE+ EDS RTU MEC TEST STRIPS



1. Dip test strip into soaking bath Virusolve+ RTU solution. and remove immediately.

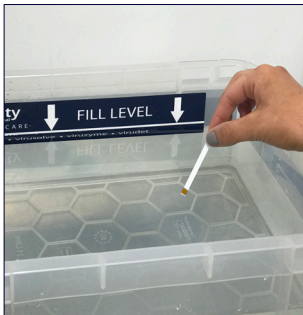

2. Shake off excess liquid and wait 15 seconds.



3. Compare colour to colour chart



Amity Limited, Libra House, West Street,
Worsbrough Dale, Barnsley, South Yorkshire,
S70 5PG, UK

T: +44 (0) 1226 770787

F: +44 (0) 1226 770757

E: sales@amityinternational.com

WWW.AIHC.CO.UK

© Amity Limited all rights reserved 2018

