

DISINFECTANTS

AMITY PAA RTU MEC TEST STRIPS



1. Dip test strip into soaking bath
PAA RTU solution for 5 seconds



2. Shake off excess liquid



3. Compare colour to colour chart

CE



Amity Limited, Libra House, West Street,
Worsbrough Dale, Barnsley, South Yorkshire,
S70 5PG, UK

T: +44 (0) 1226 247660

F: +44 (0) 1226 291936

E: sales@amityinternational.com

WWW.AIHC.CO.UK

© Amity Limited all rights reserved 2018